




# Versatus™

Intravenous catheter

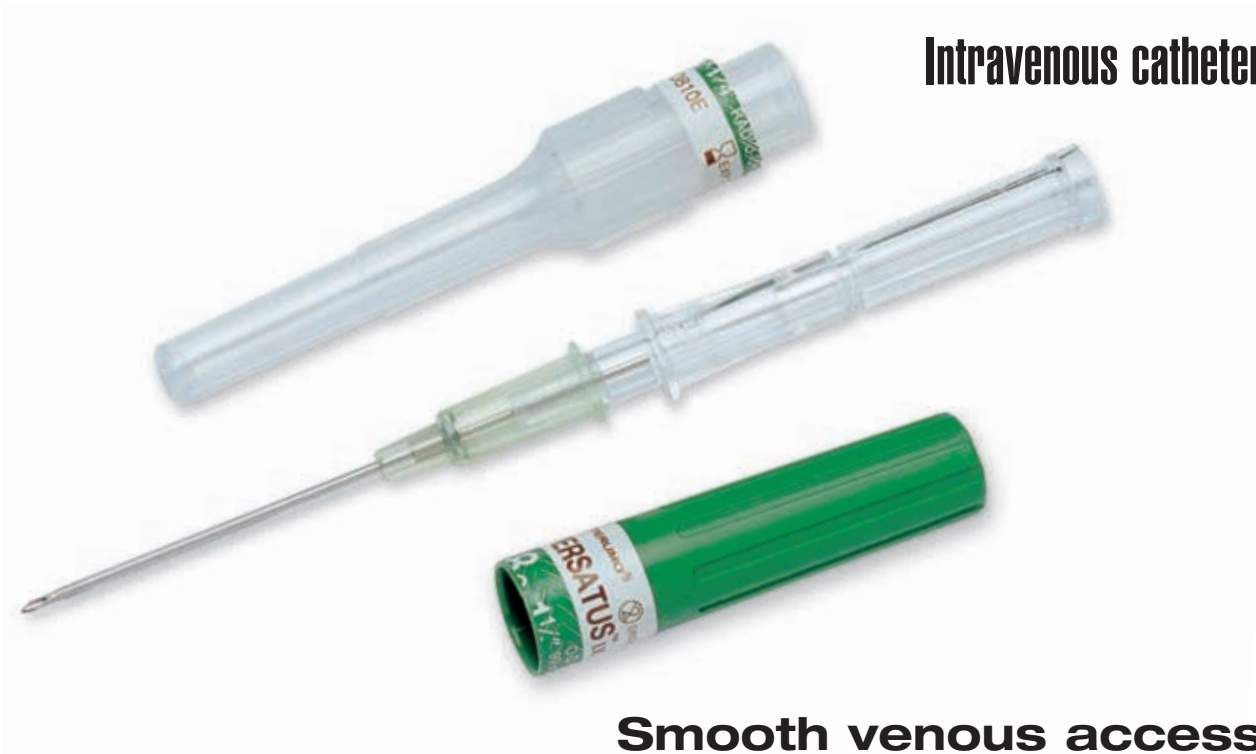

**Smooth venous access  
Better comfort for the patient**



### Superior catheter and needle tip design

The point of the introducing needle is carefully cut and honed to an ultra-sharp triple bevel that minimizes patient discomfort and tissue trauma. The gently tapered catheter tip aligns with the needle bevel to provide smooth transition from

needle to catheter during insertion. Additionally the production of Versatus™ IV Catheter is computer-controlled to maintain a constant, minimum distance from needle heel to catheter tip. This permits a smooth insertion even into small vessels while reducing the risks of penetrating the posterior wall of the vein.

### Material

To ensure minimum discomfort for the patient this catheter is made of polyurethane (PUR) which is firm at room temperature and softer at body temperature, kink resistant and biocompatible.

## Convenient rigid, colour-coded case

Each Versatus™ IV catheter is individually packed in a sterile, easy-to-open plastic case. This rigid plastic case provides added protection to both the catheter and the user. The case is colour-coded for easy identification of the catheter gauge and has a tamper-evident seal for additional safety.



## Needle bevel indicator

The mark on the upper surface of the flashback chamber serves as a needle bevel indicator. This permits precise bevel alignment even after venipuncture has been initiated.



## Rapid flashback confirmation

The flashback chamber is fully transparent for immediate confirmation of venipuncture. The incorporated hydrophobic membrane filter allows air to vent easily with virtually no resistance, which results in a rapid blood return even in the smaller gauge sizes.



## Obturator

A luer-lock obturator is available for each catheter gauge (except 24G). This obturator efficiently prevents thrombus formation in the lumen of the catheter when the infusion is interrupted. The obturator is colour-coded in accordance with the catheter to avoid confusion.



Versatus™ References	EN/ISO colour code	External catheter diameter		Internal catheter diameter mm	Catheter length mm	Flow rate ml/min	Obturator
		Gauge	mm				
SR*OF2419	Yellow	24	0.67	0.47	19	15	-----
SR*OF2225	Blue	22	0.85	0.60	25	38	SO-2225F
SR*OF2032	Pink	20	1.10	0.80	32	67	SO-2032F
SR*OF2051	Pink	20	1.10	0.80	51	59	SO-2051F
SR*OF1832	Green	18	1.30	0.95	32	99	SO-1832F
SR*OF1851	Green	18	1.30	0.95	51	88	SO-1851F
SR*OF1651	Grey	16	1.70	1.30	51	184	SO-1651F
SR*OF1464	Orange	14	2.17	1.73	64	282	SO-1464F



Every day,  
as healthcare  
professionals,  
you are working  
to ensure life  
and to contribute to  
the well-being  
of your patients.

Every day,  
**Terumo Europe**  
provides you with  
the clinical systems,  
means and solutions  
to keep life flowing.



We keep life flowing

# TERUMO®

### TERUMO CORPORATION

44-1, 2-chome  
Hatagaya  
Shibuya-ku  
Tokyo 151  
JAPAN  
Tel. +81/3 3374.8111  
Fax +81/3 3374.8196

### TERUMO EUROPE N.V.

Researchpark Zone 2 Haasrode  
Interleuvenlaan 40  
B-3001 Leuven  
BELGIUM  
Tel. +32/16.38.12.11  
Fax +32/16.40.02.49

### TERUMO EUROPE N.V. BENELUX SALES DIVISION

Researchpark Zone 2 Haasrode  
Interleuvenlaan 40  
B-3001 Leuven  
BELGIUM  
Tel. +32/16.38.14.02  
Fax +32/16.38.15.55  
THE NETHERLANDS  
Tel. +0800/023.19.38  
Fax +0800/022.34.89

### LABORATOIRES TERUMO FRANCE S.A.

1, Parc Ariane  
Bâtiment Uranus  
Rue Hélène Boucher  
73284 Guyancourt Cedex  
FRANCE  
Tel. +33/1 30.96.13.00  
Fax +33/1 30.43.60.85

### TERUMO (DEUTSCHLAND) GmbH

Lyonerstrasse 11 a  
60526 Frankfurt am Main  
GERMANY  
Tel. +49/69.66.44.20  
Fax +49/69.66.66.826

### TERUMO EUROPE SUCURSAL

**EN ESPAÑA**  
Avenida de Burgos 16-D  
Edificio EUROMOR  
Madrid 28036  
ESPAÑA  
Tel. +34/91 383.90.69  
+34/91 383.91.61  
Fax +34/1 767.04.60

### TERUMO EUROPE ROME BRANCH

Via Simone Martini 143/145  
I-00142 Roma  
ITALIA  
Tel. +39/06.519.61.420  
Fax +39/06.503.04.07

### TERUMO EUROPE N.V.

Belgien filial Sverige  
Nya Varvet  
Byggnad 90  
SE-423 71 Västra Frölunda  
SWEDEN  
Tel. +46/31.748.58.80  
Fax +46/31.748.58.90

### TERUMO U.K.

(Branch of Terumo Europe N.V.)  
3 Unity Grove  
Off School Lane  
Knowsley Industrial Park South  
Knowsley  
Merseyside  
L34 9GT  
UNITED KINGDOM  
Tel. +44/151.482.4800  
Fax +44/151.482.4811

®: Registered Trademark  
Published by Terumo Europe N.V.

[www.terumo.com](http://www.terumo.com)  
[www.terumo-europe.com](http://www.terumo-europe.com)